

To assemble, place on a firm flat surface at a comfortable working height

1



Position the lid ensuring that the word 'FRONT' faces the handle and label ...

2



... now place the lid onto the base ensuring that the inner seal of the lid is inserted into the base ...

3




... NOW press all four corners in turn to secure the lid to the base. Double check that the side latches are also engaged

4 clicks and you're there!



Follow steps 1 to 4 for safe closing and final locking of the container

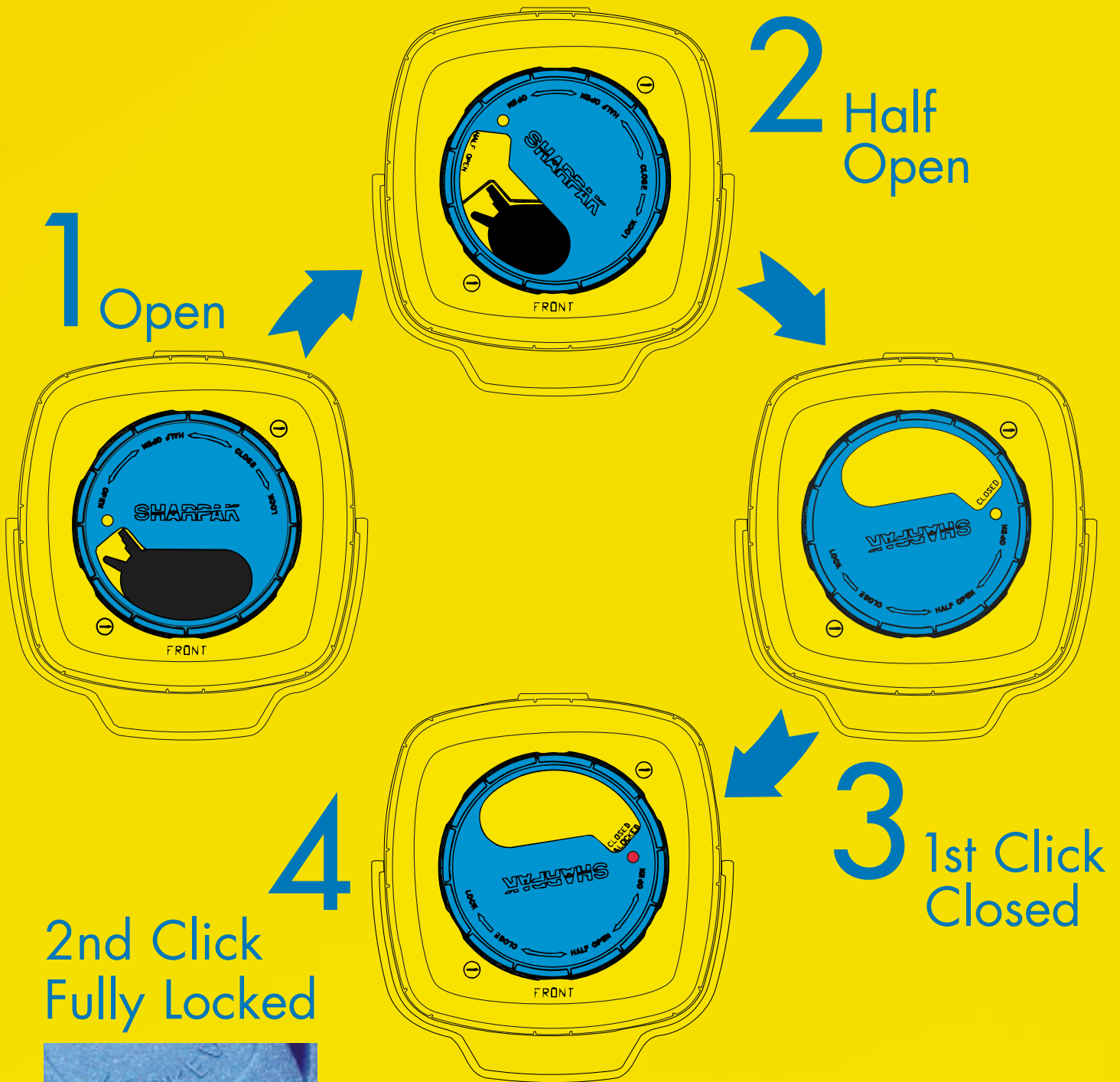

VisiLOCK™
PATENTED

For details regarding SHARPAK distributors and world wide availability, please contact: Amcor Flexibles/SPS Laboratoires
Winterbourne Road Stoke Gifford BRISTOL BS34 6PT
Tel +44 (0)117 983 6003
Fax +44 (0)117 983 6001
E mail david.scarrow@amcor-flexibles.com
www.sharpakforsharps.co.uk

AMCOR