



DISPOSAtlatPAK® has been developed by Sharpak Healthcare as a safe, cost effective and legal means of collecting and transporting clinical waste classified under UN3291.

Available in 30 and 50 litre sizes, these UN type approved containers are supplied in flat pack form ensuring storage space is kept to a minimum.

Each container is supplied with a liner bag which has the added features of a fragrance and antimicrobial additive blended in to the material. Two security ties are also included with each container to close the bag and container prior to collection and disposal.

The use of large type approved waste containers (700 litres and above) is not always practical due to space constraints in smaller collection vehicles. There is also the situation where clinical staff working in the community have the need for a safe and simple clinical waste container which can be easily transported by car.

The packs are printed in accordance with the information required to ensure conformance to the provisions of the United Nations Recommendations on the Transportation of Dangerous Goods, Chapter 9, UN 3291, Clinical Waste, Unspecified, NOS.

Subject to the quantities required it will also be possible to 'customer own label' the DISPOSAtlatPAK with the company logo and contact details.

DISPOSAtlatPAK offers the following features:

UN TYPE TESTED AND APPROVED

POLYETHYLENE LINER BAG

FRAGRANCE ADDITIVE IN LINER BAG

ANTI-MICROBIAL ADDITIVE IN LINER BAG

QUICK & SIMPLE TO ASSEMBLE

MINIMUM STORAGE SPACE REQUIRED DUE TO

FLAT PACK PRESENTATION

DISTINCTIVE COLOUR AND LABELLING

COST EFFECTIVE

REDUCED ENERGY REQUIRED FOR DISPOSAL

SECURITY TIES PROVIDED

SEQUENTIAL NUMBER AND BARCODE SYSTEM FOR FULL

DELIVERY AND COLLECTION TRACEABILITY

PURPOSE DESIGNED FREESTANDING/WALL MOUNTED STAND

CUSTOMER OWN LABEL AVAILABLE SUBJECT TO QUANTITY

The product:

DISPOSAtlatPAK 30 & 50 UN type approved containers are supplied with a polyethylene bag liner and two security ties. The pack is simple to assemble with a crash-lock base and only requires the polyethylene bag liner to be placed into position. Once the container is filled, the liner bag is closed using one of the security ties provided. The lid of the DISPOSAtlatPAK is then closed and secured with the remaining security tie.

DISPOSAflatPAK 30 & 50 litre clinical waste containers are supplied with a polyethylene liner bag which has antimicrobial and fragrance additives incorporated. Once the containers are filled the inner bag is closed using one of the two attached security ties. The closing procedure is completed by securing the lid using the remaining tie.

A purpose designed stand (picture A) has been developed for the DISPOSAflatPAK 30 & 50 litre containers. The stand can be used as a free standing unit fitted with either fixed feet or lockable castors or alternatively it can be wall mounted. The adjustment necessary to change from the 30 to the 50 litre container is achieved by the re-positioning of the support cradle. The stands are supplied in flat pack form and are quick and simple to assemble. The metal parts are finished in a combination of powder coated polyester paint and chrome plate for maximum protection against corrosion.

The accessory pack (picture B) allows you to attach surgical gloves, wipes and a sharps container.

A wall mounted basket (picture C) and laboratory trolley (picture D) are also available.



All DISPOSAflatPAK's are supplied with a sequentially numbered bar-coded label which is also used to retain the two security ties to the outside of the containers.

DISPOSAflatPAK INFORMATION

	DISPOSA flatPAK 30	DISPOSA flatPAK 50	Stand 30 &50 (pic A)	Accessory Pack (for stand) (pic B)	Laboratory Trolley (pic C)	Wall Basket (pic D)
Height	440 mm	675 mm	870 mm			
Width	295 mm	295 mm	370 mm			
Depth	295 mm	295 mm	450 mm			
Capacity	30 litres	50 litres				
UN Numbers	4G/Y9/S/**/GB/4400	4G/Y17/S/**/GB/4401				
Pack Quantity	10	10	1	1	1	1
Re-order codes	CWO5920	CWO5921	CWO5922	CWO5923	CWO5924	CWO5925



A



B



C



D

For details regarding SHARPAK distributors and world wide availability, please contact: Amcor Flexibles/SPS Laboratoires
Winterbourne Road Stoke Gifford BRISTOL BS34 6PT
Tel +44 (0)117 983 6003
Fax +44 (0)117 983 6001
E mail david.scarrow@amcor-flexibles.com
www.sharpakforsharps.co.uk