



Sharpak COMPACT, 18 and 35 HYDRI sharps containers

In line with the publication of the Department of Health Technical Memorandum HTM-07-01 titled "The Safe Management of Healthcare Waste" Amcor Flexibles SPS will continue to offer the Sharpak HYDRI range sharps containers which already conforms to the new colour code recommendations as detailed in the new guide.

All Code PURPLE Sharpak HYDRI sharps containers feature PURPLE coloured closures and labels and are recommended for the segregation, collection and disposal of sharps waste which are contaminated with cytotoxic or cytostatic medicines.

Code PURPLE Sharpak HYDRI sharps containers **MUST** only be disposed of by incineration

All Code PURPLE Sharpak HYDRI sharps containers are approved to BS7320:1990 and tested and certified in accordance with the provisions of the United Nations recommendations on the Transportation of Dangerous Goods, Chapter 9, UN3291 Clinical Waste, Unspecified, NOS.

The majority of drug manufacturers do not mark their products with the appropriate packaging group classifications, thereby leaving medical staff to decide how to safely dispose of their contaminated sharps.

What's more, when different drug types are used in the clinical environment, it is a certainty that a cocktail of drugs will result, making a correct assessment of their classification even more difficult.

The Code PURPLE range of Sharpak HYDRI containers are each supplied with a special HYDRI mat which safely absorbs any free fluid which accumulates in the bottom of the container thereby preventing any spillage during use, transportation and disposal.

The Code PURPLE Sharpak HYDRI range of sharps containers offers the following features:-

MANUFACTURED, TESTED AND CERTIFIED TO BS7320:1990

UN TYPE TESTED AND CERTIFIED

PURPLE COLOUR CODED CLOSURES TO MEET THE RECOMMENDATIONS ON WASTE SEGREGATION AS DETAILED IN THE DEPARTMENT OF HEALTH'S TECHNICAL MEMORANDUM HTM-07-01 – SAFE MANAGEMENT OF HEALTHCARE WASTE

SECURE CLOSURE WITH LOCKING FACILITY

VISILOCK INDICATOR IS FITTED ON THE SHARPAK COMPACT HYDRI, 18 HYDRI AND 35 HYDRI

DISTINCTIVE COLOURING AND CLEAR LABELLING

SEQUENTIAL NUMBERING FOR TRACEABILITY

TOUGH AND DURABLE

SHARPAK COMPACT HYDRI AND SHARPAK 18 HYDRI SUPPLIED ASSEMBLED

SHARPAK 35 HYDRI – QUICK AND EASY TO ASSEMBLE

COST EFFECTIVE

PURPOSE DESIGNED WALL AND TROLLEY BRACKETS, STANDS AND KICK TROLLEYS

SUPPLIED WITH SHARPAK HYDRI MATS TO ABSORB ANY SMALL AMOUNTS OF FREE LIQUID IN THE EVENT OF AN ACCIDENTAL SPILLAGE IN THE CONTAINER

SHARPAK™
HEALTHCARE



VisiLOCK™
PATENTED

FOR THE SAFE DISPOSAL OF CYTOTOXIC AND CYTOSTATIC CONTAMINATED SHARPS WASTE


Following the research work undertaken, we concluded that the size of HYDRI container used should, where possible, be kept as small as practical based on the activity level of the department.

By selecting the most suitable size of container this will help to avoid excessive amounts of toxic substances accumulating in the container.

The following sizes of Sharpak HYDRI containers are available:- HYDRI COMPACT, HYDRI 18, HYDRI 35, HYDRI 60, HYDRI 120 and HYDRI 220.

SHARPAK® PRODUCT RANGE

PRODUCT	MAX CAPACITY	CAPACITY TO FILL LINE	Re-order code	HEIGHT	WIDTH	DEPTH
Sharpak COMPACT HYDRI	0.5 litres	0.39 litres	SU06215	186 mm	67 mm	67 mm
Sharpak 18 HYDRI	1.58 litres	1.27 litres	SU06604	210 mm	104 mm	104 mm
Sharpak 35 HYDRI	3.5 litres	2.63 litres	SU03510	195 mm	177 mm	206 mm

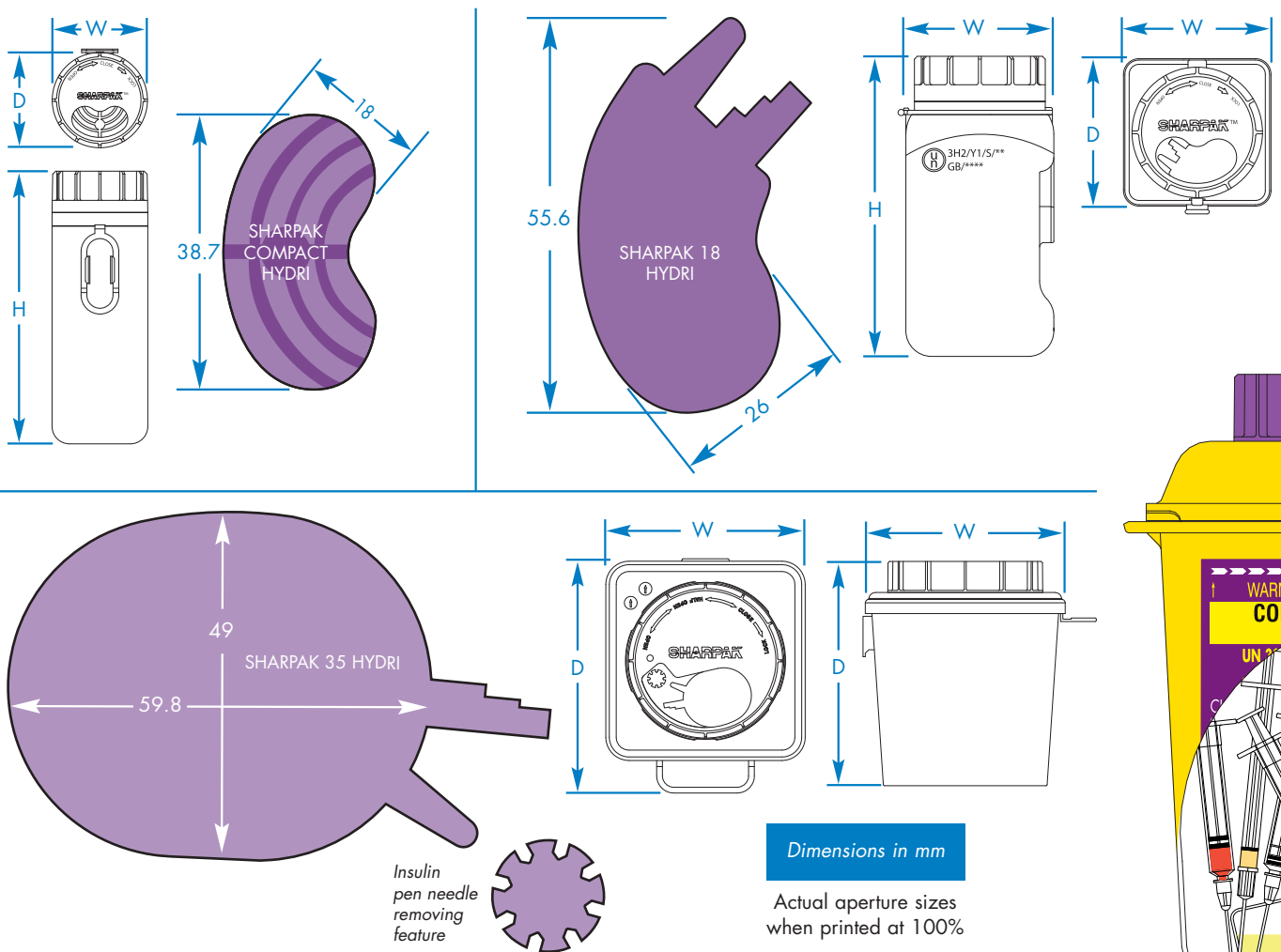

For further information on any of these Sharpak sharps containers see the product specific data sheets.

For details regarding SHARPAK distributors and world wide availability, please contact:
 Amcor Flexibles/SPS Laboratoires Winterbourne Road Stoke Gifford BRISTOL BS34 6PT
 Tel +44 (0)117 983 6000 Fax +44 (0)117 983 6001
www.sharpakforsharps.co.uk