

# Closure Procedure



To assemble, place on a firm flat surface at a comfortable working height

1



Position the lid ensuring that the word 'FRONT' faces the handle and label ...

2



... now place the lid onto the base ensuring that the inner seal of the lid is inserted into the base ...

3



... NOW press all four corners in turn to secure the lid to the base. Double check that the side latches are also engaged

4 clicks and you're there!

**SHARPAK**<sup>TM</sup>  
HEALTHCARE

For details regarding SHARPAK distributors and world wide availability, please contact: Amcor Flexibles/SPS Laboratoires  
Winterbourne Road Stoke Gifford BRISTOL BS34 6PT  
Tel +44 (0)117 983 6003  
Fax +44 (0)117 983 6001  
E mail [david.scarrow@amcor-flexibles.com](mailto:david.scarrow@amcor-flexibles.com)  
[www.sharpakforsharps.co.uk](http://www.sharpakforsharps.co.uk)

